

Ultimate Safety, Inc.

21222 Gathering Oak, Ste. 103, San Antonio, TX 78260
Phone: 1-800-248-9241 Fax: 1-866-858-4623

BedCall Inservice – Competency Check List

- ___ Date written on UltiMat.
- ___ UltiMat placed on top of mattress, under pelvic area of patient and under all bed linens.
- ___ If utilizing air mattress or egg crate, place UltiMat under these items.
- ___ UltiMat must lay flat! No bent over edges.
- ___ UltiMat secured to bed frame utilizing rubber bands and clips.
- ___ Clip the clip to the rubberband itself. DO NOT CLIP TO MATTRESS!
- ___ UltiMat is centered on mattress; ensuring laying flat on mattress.
- ___ Plug UltiMat into BedCall Monitor.
- ___ Plug Nurse Call Cord into adapter.
- ___ Plug BedCall Monitor into wall.
- ___ Hang BedCall Monitor from the bed frame secured tightly with velcro strap (do not strap to the side rails) or hang BedCall Monitor in Holster on headboard of bed.
- ___ Test the UltiMat by:
 - Placing your hand on the UltiMat.
 - Turn BedCall Monitor on to Monitoring.
 - Check the time delay for appropriate setting.
 - Remove your hand and you should get alarm according to the above time delay selected.
- ___ EdgeSense Feature:
 - Do you want to utilize this feature on this patient?
 - If yes, turn on the EdgeSense.

After above competencies have been achieved, you are now ready to utilize the BedCall Fall Prevention System. Please remember from the inservice, that you, the staff, are essential to quick response for an effective fall prevention system to be effective.